LC CALCIUM HYDROXIDE LINER

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Innovative dental system

LC CALCIUM HYDROXIDE LINER

Light cure radiopaque liner with calcium hydroxide

Instructions for Use

MD

Product description

LC CALCIUM HYDROXIDE LINER is a light cure, radiopaque liner with calcium hydroxide.

Indications/Intended use

- indirect pulp capping
- lining of cavities for following filling procedures

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor let the material come into contact with materials containing eugenol.

Application

Clean and dry the cavity thoroughly. Avoid contamination

Screw off the cap of the syringe and substitute it by a NeedleTip. After use discard the NeedleTip and attach the cap to the syringe again. NeedleTips are for single use only.

Note:

In direct proximity to the pulp (direct pulp capping) application of an aqueous calcium hydroxide liner is recommended.

To enhance bonding of LC CALCIUM HYDROXIDE LINER use a bonding agent. Etching and application of a suitable light cure bonding agent according to the corresponding instructions is recommended. In areas with a thin dentin layer over the pulp do without bonding (pulp protection).

Apply LC CALCIUM HYDROXIDE LINER in a thin layer (max. 1 mm) and light cure for **40 seconds** by using a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm².

Note: Avoid placing LC CALCIUM HYDROXIDE LINER on enamel or the margins of the cavity.

After curing apply the filling according to manufacturer instructions.

Storage

Store at 4 - 25 °C (39 - 77 °F) protected from light and moisture. Close syringe immediately after use. Do not use after expiry date.

Additional Notes/Warnings

- When using LC CALCIUM HYDROXIDE LINER wear protective glasses and cover the patient eyes.
- Unpolymerized materials may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- In case of contact with eyes, rinse with copious amounts of water and consult a physician immediately.
- Avoid contact with skin. If accidental contact occurs, wash immediately with water.
- Do not use any resin to adjust viscosity of LC CALCIUM HYDROXIDE LINER.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior Dental Materials GmbH product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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