LC HI-FLOW NANO

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LC HI-FLOW NANO

Instructions for Use

Low shrink low abrasion flowable light cure nano-composite with improved mechanical properties

MD

Product description

LC HI-FLOW NANO is a visible light cure high-viscous flowable **nano-composite for anterior and limited posterior restorations**. Beside its low shrink and low abrasion LC HI-FLOW NANO shows improved mechanical properties.

LC HI-FLOW NANO contains dimethacrylate based resins and inorganic fillers. Radiopacity of LC HI-FLOW NANO is 140 % aluminium (radiopacity of enamel: ~200 % aluminium, of dentine: ~100 % aluminium).

The composite material meets the requirements of **DIN EN ISO 4049**, type 1, class 2, group 1.

Indications/Intended use

- Restorations of class III, IV and V
- Small class I and II restorations
- Lining of cavities

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

Application

Preparation of Tooth and Color Matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and color matching.

Ascertain the tooth shade while teeth are still moist and select the appropriate LC HI-FLOW NANO shade.

LC HI-FLOW NANO is shaded according to VITA®-Shades.

Cavity Preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction.

For pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

Condition and apply a bonding agent according to the manufacturer instructions. The total etch technique with an etching gel and subsequent application of a suitable light cure bonding is recommended. Otherwise use a suitable self etching bonding agent before application of the LC HI-FLOW NANO.

Placement of LC HI-FLOW NANO

Apply LC HI-FLOW NANO in the selected shade. Place it by using a suitable instrument. Transparent matrix strips may be used.

Application from syringe:

Screw off the cap of the syringe and substitute it by one of the supplied NeedleTips. After use discard the NeedleTip and attach the cap to the syringe again. NeedleTips are for single use only due to hygienic reasons.

Curing of LC HI-FLOW NANO

For an optimum result apply **LC HI-FLOW NANO** in layers of **max. 2 mm**. Light cure each increment separately.

By using a polymerization unit (wavelength range 400-500 nm) with a **light intensity of at least 1000 mW/cm²**, cure each increment as follows:

Lighter shades (e.g. A1, A2, B2) **20 seconds**

Darker shades (e.g. A3.5) **30 seconds**

Hold the light emission window as close as possible to the filling material.

Finishing

Depending on the shape of the area contour with finishing diamonds, flexible grinding discs or carbide burs. Polish to high gloss with polishing discs or polishing brushes.

Storage

Do not store above 25°C. Protect from direct sunlight. Do not use after expiry date.

Close syringe immediately after use to avoid exposure to light.

Additional Notes/Warnings

- The ambient light of the dental lamp may start polymerization of the composite.
- Do not use any resin to adjust viscosity of the composite restorative.
- Avoid contact with skin, mucous membrane and eyes.
- Unpolymerized composite may have an irritant effect and can lead to sensitization against methacrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- VITA® is a registered trade mark of the VITA-Zahnfabrik, Bad Säckingen, Germany.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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P.L. Superior Dental Materials GmbH, Stiller Weg 15 A, 22607 Hamburg, Germany