# LC MH CERAMIC NANO

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# LC MH CERAMIC NANO

Instructions for Use

Light cure high performance composite with excellent high gloss polishability, for Universal Opacity restorations

MD

## Product description

LC MH CERAMIC NANO is a light cure radiopaque composite for anterior and posterior restorations with permanent gloss and toothlike opalescence. LC MH CERAMIC NANO shows excellent physical properties and good handling features. High gloss polishability demands only little effort. LC MH CERAMIC NANO contains dimethacrylate based resins and inorganic fillers < 0.2 µm. The total filler content is 80 % by weight and 76 % by volume.

Radiopacity of LC MH CERAMIC NANO is 200 % aluminium (radiopacity of enamel: ~200 % aluminium, of dentine: ~100 % aluminium). LC MH CERAMIC NANO meets the requirements of **ISO 4049**, type 1, class 2, group 1.

## Indications/Intended use

- Restorations of all cavity classes
- Direct veneers
- Repair of ceramic restorations (e.g. veneers, crowns)

## Performance features

The performance features of the product meet the requirements of the intended use.

## Contraindications

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

## Patient target group

Persons who are treated during a dental procedure.

## Intended users

This medical device should only be used by a professionally trained dental practitioner.

## Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

## Application

### Preparation of Tooth and Color Matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and color matching.

Ascertain the tooth shade while teeth are still moist and select the appropriate LC MH CERAMIC NANO shade.

LC CERAMIC NANO shades correspond with the VITA® scale and have different opacities.

### Cavity Preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction.

For pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

Condition and apply a bonding agent according to the manufacturer instructions. The total etch technique with an etching gel and subsequent application of a suitable light cure bonding agent is recommended. Otherwise use a suitable self etching bonding agent before application of the LC MH CERAMIC NANO.

### Application and Curing of LC MH CERAMIC NANO

Apply LC MH CERAMIC NANO in the selected shade. LC MH CERAMIC NANO is easy to model. Apply the restorative material in layers and light cure.

In case of deep and extended cavities best results are obtained with a build-up of a base liner using a light cure resin modified cement or a light cure flowable composite, followed by the application of LC MH CERAMIC NANO.

In case of small cavities LC MH CERAMIC NANO can be applied directly.

For an optimum result apply LC MH CERAMIC NANO in **layers of max. 2 mm**. Light cure each increment separately.

By using a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm<sup>2</sup>, cure each increment separately.

Shades	Incremental depth	Cure time
Body, Enamel, Translucent	2,0 mm	20 seconds
Dentine	1,5 mm	40 seconds

Hold the light emission window as close as possible to the filling material.

## Finishing

Depending on the shape of the area contour with finishing diamonds, flexible grinding discs or carbide burs.

Polish to high gloss with polishing discs or polishing brushes.

## Storage

Do not store above 25°C (77°F). Protect from direct sunlight. Do not use after expiry date.

Close syringe immediately after use to avoid exposure to light.

## Additional Notes/Warnings

- The ambient light of the dental lamp may start polymerization of the composite.
- Do not use any resin to adjust viscosity of composite restorative material.
- Avoid contact with skin, mucous membrane and eyes.
- Unpolymerized composite may have an irritant effect and can lead to sensitization against methacrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- VITA® is a registered trade mark of the VITA-Zahnfabrik, Bad Säckingen, Germany.
- Keep away from children!

## Disposal

Disposal of the product according to local authority regulations.

## Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

## Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior Dental Materials GmbH product.

## Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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P.L. Superior Dental Materials GmbH, Stiller Weg 15 A, 22607 Hamburg, Germany