Produktbeschreibung

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arkom@arkom-org.com

Product description

UNI BOND is a simple to use light cure universal dental adhesive. UNI BOND can be used as a self-etch adhesive as well as with preceding selective enamel etching or with the total-etch technique on enamel and dentin for better adhesive strength. UNI BOND can be used as a light cure adhesive for moist, wet and dry surfaces. Based on innovative MDP technology UNI BOND allows an excellent adhesion.

For applications, where light cure is difficult or impossible (e.g. in the root canal) UNI BOND has to be used in combination with CATALYST UNI BOND. Please refer to the instructions for CATALYST UNI BOND.

Indications/Intended use

Adhesive for the self-etch technique, selective enamel etching or total-etch technique for:

- Direct light cure composite and compomer restorations.
- Core build-ups with light, dual and self cure composites if light cure of the adhesive is possible.
- Cementing of inlays, onlays, crowns and bridges with light, dual and self cure composite cements if light cure of the adhesive is possible.
- Repair of fractured composite restorations

Use UNI BOND only with dual cured or self cured composites if light cure of the adhesive is guaranteed.

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor let the material come into contact with materials containing eugenol.

Application

1. Isolation

Rubber dam is the recommended method of isolation.

2. Cavity Preparation

Prepare the cavity according to the principles of the adhesive restoration technique.

3. Pulp Protection

For pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

4. Enamel and dentine conditioning (optional)

Selective enamel etching or etching by application of the total-etch-technique on enamel and dentin can be used to improve the bonding.

Note:

Unprepared enamel surfaces must always be conditioned with etching gel.

Etching precaution:

It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as described below. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

a. Selective enamel etching

Apply etching gel onto the enamel surfaces and leave in place for **at least 15 seconds**. Deciduous teeth are etched correspondingly longer. Rinse for 20 seconds with water and dry it in a water-free and oil-free airstream. The etched enamel surface should have a chalky white appearance.

b. Total-Etch Technique

Apply etching gel onto the enamel and dentine surfaces beginning with the enamel bevels. Condition **the enamel for at least 15 seconds and the dentine for 15 seconds**. (This results in 20–30 seconds etching of enamel and 15 seconds of dentine). Deciduous teeth are etched correspondingly longer. Rinse for 20 seconds with water. Dry it in a water-free and oil-free airstream, but **do not desiccate**. The etched enamel bevel should have a chalky white appearance.

5 Application of UNI BOND

Dispense one or two drops UNI BOND into a mixing well and use immediately after dispensing. **Close bottle immediately after use.**

Apply UNI BOND with a brush onto the whole enamel and dentine surface in a thin and even layer. Keep the adhesive agitated for at **least 20 seconds**. Applying the adhesive on the tooth surface without agitation is inadequate. The material should build a homogeneous layer.

Air thin gently (**at least 5-10 seconds**) to remove the volatile components and to disperse the adhesive until a glossy and uniform layer results. Then light cure with a suitable dental halogen light unit or an LED (wavelength 400–500 nm, light intensity min. 1000 mW/cm²) for **20 seconds**.

6. Application of restorative materials

Refer to manufacturer instructions for placement of the light, dual or self cure restorative material.

Note for direct restorations:

Best results are obtained with application of a thin layer of a light cure flowable composite followed by the application of a moldable composite. Light cure each composite layer separately according to the corresponding user instructions.

Additional informations/Warnings

- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

Storage

Do not store above 25 °C (77 °F)! Protect from direct sunlight. Do not use after expiry date.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior Dental Materials GmbH product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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